## Smilekey How-to-Quick Guide


Smilekey Tip is designed to be positioned 10mm from the center of coping.



We recommend to move tip from soft tissue to occlusion during activation. Repeat activation on opposite side for another 4 seconds if necessary.

8 seconds (2 back-to-back) for Zirconia, eMax and procelain prosthesis.

Metal objects, such as retractors, mirrors, etc., should not be near during Smilekey activation. Smileloc icon indicates the activation side.

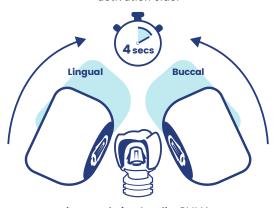

4 seconds for Acrylic, PMMA, Resin prosthesis.


Angle tip toward the center of coping. Avoid pointing at implant.



Please refer to IFU0004 for Full Product Information

## Smilekey How-to-Quick Guide

Smileloc icon indicates the activation side.

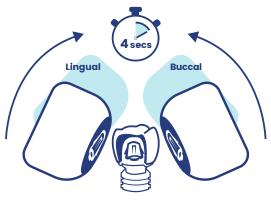

- Angle tip toward the center of coping. Avoid pointing at implant.
- We recommend to move tip from soft tissue to occlusion during activation. Repeat activation on opposite side for another 4 seconds if necessary.
- Metal objects, such as retractors, mirrors, etc., should not be near during Smilekey activation.



Smilekey Tip is designed to be positioned 10mm from the center of coping.

- 4 seconds for Acrylic, PMMA, Resin prosthesis.
- 8 seconds (2 back-to-back) for Zirconia, eMax and procelain prosthesis.





Please refer to IFU0004 for Full Product Information